# smartLAB<sup>®</sup>

hrm W

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# smartLAB<sup>®</sup>

## hrm W

*Heart rate monitor with Bluetooth Smart & ANT+*

## User Manual

*Please read this user manual thoroughly before first using this device*

*Current information about this device, visit smartLAB on  
[www.support.hmm.info](http://www.support.hmm.info)*

HMM Diagnostics GmbH  
D-69221 Dossenheim, Germany  
[www.hmm.info](http://www.hmm.info)

\*The Bluetooth<sup>®</sup> word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by HMM Diagnostics is under license. Other trademarks and trade names are those of their respective owners.

# Content

<b>I. Introduction</b>	<b>17</b>
<b>II. Device Overview</b>	<b>18</b>
Heart rate monitor	18
Adjustable elastic belt	19
<b>III. Setup &amp; Functions</b>	<b>20</b>
Unpacking the product	20
How to wear chest strap	20
Maintenance	22
Insert Batteries	23
Important	24
Supported Devices	24
Regularity and Safety Notice	25
Warranty	27

## I. Introduction

Thank you for using the **smartLAB® hrm W heart rate monitor**. This product is a sensor type heart rate monitor that will detect heart rate and send your data to a ANT+ or *Bluetooth®* Smart receiver, such as PC, Tablet or Smartphone.

**IMPORTANT:** This product is for sport purposes only and it is not meant to replace any medical device.

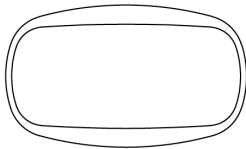
This manual contains important safety and care information, and provides step-by-step instructions for using this product. Read the manual thoroughly, and keep it in a safe place in case you need to refer to it later.

**IMPORTANT:** This product is not to be coupled like the usual standard Bluetooth devices. You need an app to see the data. If the app supports these devices, you can see the values without coupling with the smart phone.

## II. Device Overview


### A. Heart rate monitor

#### a. Front View

*pict. 1*

#### b. Rear View

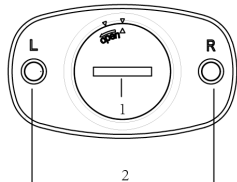
*pict. 2*

1. Left & Right wearing directions

2. Battery Door

## B. Adjustable elastic belt

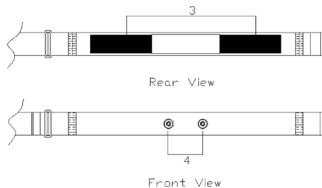*pict. 3*

3. Electrodes

4. Fastening buttons

### III. Setup and Functions

#### 1. Unpacking the product

In this box, you will find:

- a. **smartLAB® hrm W** heart rate monitor (transmitter)
- b. Adjustable elastic belt
- c. CR2032 battery – already installed
- d. User manual


#### 2. To wear chest strap

- a. Wet the conductive pads on the rear of the chest belt with a few drops of water, Saliva or conductive gel to ensure solid contact.


*pict. 4*

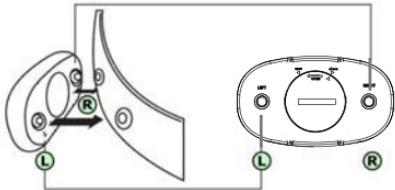b. Attach the transmitter to the strap

*pict. 5*

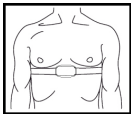c. Strap the heart rate chest belt across your chest. To ensure an accurate heart rate signal, adjust the strap until the belt fits snugly below your (chest) pectoral muscles.

*pict. 6*

C. Run software on smart phone, to receive the heart rate data.

## Maintenance

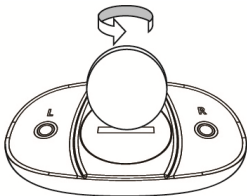
- Cleaning of transmitter and strap in a washing machine or dryer is strongly prohibited
- Before hand washing the device, detach the transmitter of Heart Rate Monitor where electrodes are located. This part is not washable
- While washing the straps, make sure water is no higher than 30°C
- Ironing, bleaching or heating is prohibited
- Do not dump the device or use force while operating it
- Avoid exposing the device to high temperatures or direct sunlight
- Let the device acclimate after having stored it in a temperature close to zero. Do not immediately use it in such a condition
- Do not try to disassemble the device
- If you do not use the device for a longer time, please take out the batteries

## Insert Batteries

The transmitter needs one CR2032 3V battery. The battery is already installed when bought. The battery typically lasts up to one year.

To replace the chest belt battery:

- First twist the compartment lid using a coin counter-clockwise until it clicks out of place.

*pict. 7*

- Remove the old battery and insert the new battery with the +-side upwards.
- Replace the battery compartment lid by twisting it clockwise until it is firmly in place.

## Important

1. Non-rechargeable and rechargeable batteries must be disposed of properly.

Heart Rate Monitor and batteries need to be disposed of according to local regulations.

2. Batteries are extremely dangerous when swallowed! Therefore, keep batteries and small articles away from children. If a battery has been swallowed seek immediate medical advice.

3. The batteries supplied must not be recharged, reactivated by any other means, dismantled, put into fire or short-circuited.

## Supported Devices

This device is an ANT+ / *Bluetooth*<sup>®</sup> Smart device, compatible and applicable with many ANT+ / *Bluetooth*<sup>®</sup> Smart receivers, like several iOS and Android Smartphones and Tablets as well as PC's

## Regularity and Safety Notice

### Declaration of Conformity

This product is compliant with the EC Directives: -1999/5/EC

### Safety Statement

This product has been verified to conform to the safety certification in accordance with the specification of EN Standards: EN 60950-1:2006 + A11:2009 + A1:2010 + A12:2011

### WEEE note

The WEEE (Waste Electrical and Electronic Equipment) Directive, that came into effect as European law on 13th February 2003, led to a major change in the disposal of electrical equipment.

The primary purpose of this directive is to prevent of electronic waste (WEEE), while encouraging for reuse, recycling and other forms of reprocessing to reduce waste.

The WEEE logo on the product and the packaging indicates that it is not allowed to dispose of the product in the ordinary household waste. It is your responsibility to deliver all the disused electrical and electronic equipment to the respective collection points. A separate collection and proper recycling of electronic waste helps dealing economically with natural resources. Furthermore, the recycling of electrical waste is a contribution to the conservation of our environment and thus the health of all people. More information concerning the disposal of electrical and electronic devices, reprocessing and the collection points you can get from the local authorities, waste disposal companies, from retailers and manufacturers of the device.

## RoHS compliance

This product complies with Directive 2002/95/EC of the European Parliament and the Council of 27th January 2003 regarding the limited use of dangerous substances in electrical and electronic equipment (RoHS) and its variations.

## Warranty

HMM Diagnostics GmbH products need to fulfill high quality requirements. Because of this reason, HMM Diagnostics GmbH gives a 2-year warranty by purchasing this **smartLAB**<sup>®</sup> product. You can even extend the warranty from 3 to 5 years without extra pay when you register your product. Please register online under following website: [www.hmm.info/en/registration](http://www.hmm.info/en/registration).

*Wear parts, batteries etc. are excluded from warranty.*

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